


# The mobile measuring board from BSN-JOBST

Simple handling for **precise results**

With the new, handy measuring board from BSN-JOBST you can now effectively simplify your daily work. It is considerably smaller and is extremely flexible to use.

When folded up, the measuring board from BSN-JOBST is so compact that, unlike static measuring systems, it can be stored or transported unobtrusively and with minimum space requirements. It is quickly on hand when you need it and is ready to use at all times.



Enclosed are two JOBST® measuring tapes for convenient measuring for medical compression stockings.

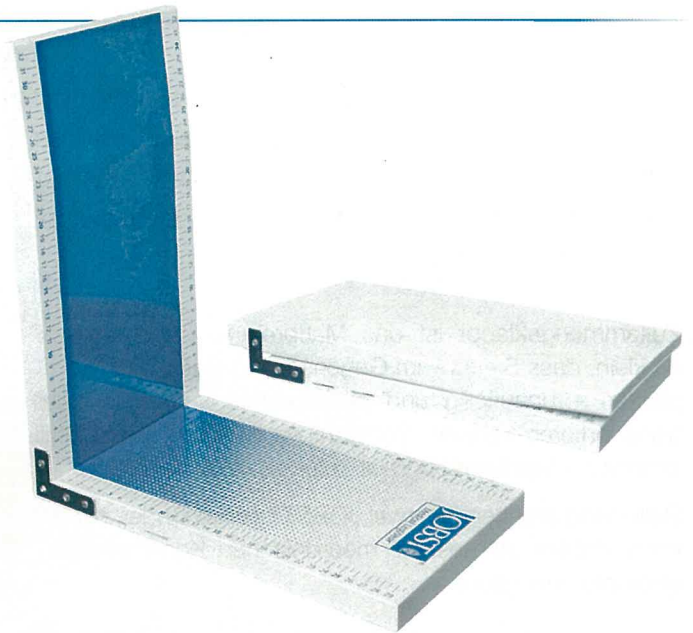

On the hinged side of the board with our JOBST® logo you can see two fixing slots on each edge for precise positioning of the measuring tapes. Insert the measuring tape into one of the openings until the metal end is no longer visible. To fix the measuring tape in place, carefully slide it to the left, to the other end of the opening. Now you can easily pull the measuring tape out to the required length. To remove it after measuring the patient, simply slide it to the right and pull it out of the opening.



Your customers can stand, sit or lie down when being measured for their compression stockings. This makes the process easier for patients with restricted mobility. It is important that the patient's calf is in contact with the all-blue side which narrows towards the top. The sole of the foot will therefore be on the part which has the JOBST® logo. Now you can read off the length of the leg on the scale of the measuring tape.

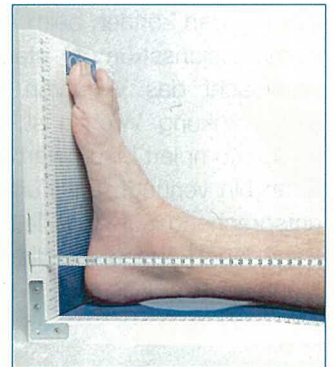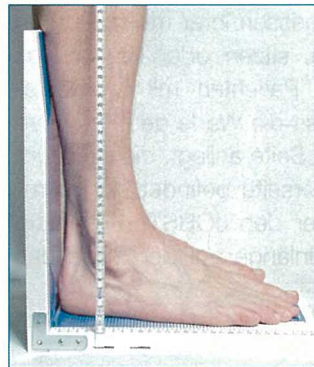

**Here at BSN-JOBST, we very much hope that our new measuring board can make your work easier and wish you continued success in providing expert care to your patients.**

BSN-JOBST GmbH  
Beiersdorfstraße 1 · 46446 Emmerich am Rhein  
Tel.: 02822 607-100 · Fax: 02822 607-199  
www.jobst.de

BSN-JOBST – a company of 

**JOBST®**

Comfort, Health and Style!