

JOBST®

Enjoy the benefits of

the new JOBST® Farrow® Toe Cap

Suitable for wearing
on either foot for
extra convenience



Left or Right

Fine seams with
ultra-thin fabric allows
air to pass to enhance
patient comfort



Breathable

The smooth
compression fabric
made from a blend of
soft yarns provides
high wearing comfort



Soft Touch

The toes can be
easily trimmed with
scissors to exactly
the required length
without unravelling



Trimmable

JOBST® Farrow® Toe Cap

The JOBST® Farrow® Toe Cap is designed for mild to moderate lymphoedema patients who require a cost effective, **ready-to-wear** option. Offered in a range of sizes, the JOBST® Farrow® Toe Cap covers a broad range of patients and can be worn independently or in combination with JOBST® Farrow® Wrap.

Donning Instructions

Open wounds should be covered before applying toe cap. Remove jewellery before applying. The toe cap can be used on either foot:

- A. **Right foot:** label is facing outward (Fig. 1)
- B. **Left foot:** turn the toe cap “inside out”, label is on the inside

Follow these steps when putting on your toe cap:

1. Insert toes and foot into the toe cap opening (Fig. 2).
2. Position the toe cap so that it comfortably fits over the toes and foot (Fig. 3).
3. Work toe cap in between your toes to web space (Fig. 4).
4. If the toe lengths are too long, mark the toe at the appropriate length, remove the toe cap, then trim the length of the toe fabric to the marked area and reapply.

Be careful not to remove too much of the toe length.



Fig. 1



Fig. 2

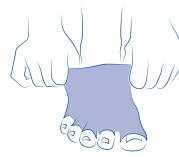


Fig. 3




Fig. 4

Compression Levels

20 - 30 mmHg

Measurement Table

Size	cA
XS	20-22 cm
S	22-24 cm
M	24-26 cm
L	26-28 cm

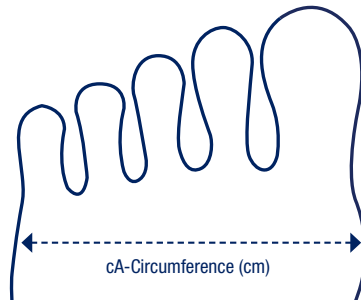

Colours



Beige

Order Information

Product Code	Size	Compression	Colour
72947-00	XS	20 - 30 mmHg	Beige
72947-01	S	20 - 30 mmHg	Beige
72947-02	M	20 - 30 mmHg	Beige
72947-03	L	20 - 30 mmHg	Beige