



Comfort, Health and Style!

JOBST® FarrowWrap®

LYMPHOEDEMA MANAGEMENT
MADE SIMPLE THROUGH ADJUSTABLE
SHORT-STRETCH WRAP SOLUTIONS



THERAPIES. HAND IN HAND.



an Essity company



HOW DO YOU STRIKE THE BALANCE BETWEEN BOTH YOUR PATIENT'S MEDICAL NEEDS AND THEIR QUALITY OF LIFE?

MORE OPTIONS FOR YOUR PATIENTS – FITTED FOR LIFE

At BSN medical, we help healthcare providers like you find the right therapy to keep patients healthy, independent, active and comfortable. We also focus on motivating patients to continue therapy for optimal treatment outcomes.

With the integration of JOBST® FarrowWrap® products we are a step further towards being able to offer you a full portfolio of products to help you deliver what you and your patients want: the freedom to live their lives.

FITTED

TO YOUR THERAPY
TO YOUR LIFESTYLE
TO YOUR EVERY DAY AND NIGHT

FOR LIFE

Managing a chronic illness such as lymphoedema over a prolonged period of time requires a dynamic approach and active patient participation. At BSN medical, we know that the ultimate goal is to empower patients to manage their condition and take control of their own care. We provide adaptable therapy options to cater to patients' varied preferences and needs, making life with lymphoedema a manageable challenge for patients.

Our easy-to-adjust JOBST® FarrowWrap® system makes daily compliance more manageable. We know that patients are not just looking for the best therapy available, but the best therapy for them, helping to fit their unique lifestyle and compression requirements without compromising their medical needs.

WHAT IS JOBST® FarrowWrap®?

HOW IT WORKS

How does a JOBST® FarrowWrap® achieve compression?

- Utilising special fabrics
- Manufactured with smart elastic short-stretch bands comprised of multi-layered fabrics
- Amount of band overlap from bottom to top
- Optimal width of bands

It is important that every band of a JOBST® FarrowWrap® garment be applied with moderate stretch. Having a consistent amount of tension throughout the garment helps to ensure graduated compression.

The elastic short-stretch fabrics provide sustained resting compression while also augmenting the calf-muscle pump to deliver high working compression.

BENEFITS


- Easy use of the simple VELCRO® system saves therapist's and patient's time and supports easy donning and doffing for maximal patient independence.
- The product is adjustable, so fluctuating edema levels can be controlled more easily by the patients themselves – empowering them to control their own care.




CHARACTERISTICS OF SHORT-STRETCH TECHNOLOGY

- Allows creation of products with different compression levels at end-stretch
- Provides superior augmentation of the body's natural muscle pump action
- Maximises calf-muscle function

- Minimises swelling throughout the day once garment is donned
- Provides higher compression when the patients stand or are active and less when they are not

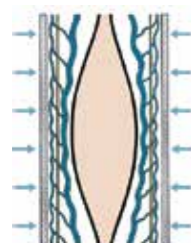

The pressure, exerted onto the body from the outside when the muscles at rest, is called "resting pressure".

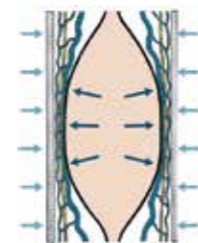

The pressure exerted onto the body from the inside as the muscle activates and expands, is called the "working pressure".

Short-stretch bandages deliver a low resting pressure and a high working pressure, producing a massage effect on the calf-muscle when the patient is active and producing a subsequent increase in venous and lymphatic return. This makes short-stretch systems very effective in the treatment of lymphoedema.

THE JOBST® FarrowWrap® PRODUCT RANGE

LITE

A double laminated fabric suitable for mild to moderate compression needs, making it especially suitable for palliative and paediatric patients. The fabric is both durable and breathable.

Available variants

- Upper and lower limb
- Ready-to-wear (RTW)

- **Compression: 20-30 mmHg**
- **Mild to moderate chronic oedema/lymphoedema**

- Conforms to limb shape
- Good choice for patients who have neuropathy or peripheral vascular disease (PVD)
- Recommended for paediatric patients
- Suitable for sensitive skin and as such palliative care patients
- Machine washable and can be tumble dried on low heat

CLASSIC

The original material that has been clinically proven over many years of use on thousands of patients worldwide. The rigid nature of the material offers performance beyond the compression range making it excellent for patients with moderate to severe compression levels.

Available variants

- Lower limb
- Ready-to-wear (RTW)

- **Compression: 30-40 mmHg**
- **Moderate to severe chronic oedema/lymphoedema**

- Suitable for oedema with skin folds and irregular limb shapes
- Durable, rigid fabric with a comfortable feel
- Reinforced spine for easier donning
- Less likely to roll down and will not work into skin folds
- Hand wash only and lay flat to dry

STRONG

Premium fabric with strong compression and soft inner layer suitable for patients with a good limb shape and moderate to severe oedema.

Available variants

- Lower limb
- Ready-to-wear (RTW)

- **Compression: 30-40 mmHg**
- **Moderate to severe chronic oedema/lymphoedema with a good limb shape**

- Less rigid than CLASSIC
- Contoured bands conform to the limb and resist sliding
- Highly breathable and durable
- Suitable for minor skin folds and distortion
- Material less likely to pill or fray
- Machine washable and can be tumble dried on low heat

MANAGING LYMPHOEDEMA CAN BE A COMPLEX JOURNEY ...

All patients are different and have different needs, therefore different solutions are sometimes needed.

There are several compression solutions to control limb swelling. Compression bandages and compression stockings are considered first line therapy, but don't work for all patients.

FITTED TO YOUR THERAPY
TO YOUR LIFESTYLE
TO YOUR EVERY DAY AND NIGHT
FOR LIFE

FIND THE BEST SOLUTION FOR YOUR PATIENT



Bandaging by healthcare professionals

During the intensive treatment programme phase known as Complex Decongestive Therapy (CDT), compression is performed using short-stretch compression bandages that are applied by a trained healthcare professional. Multi-layer lymphoedema bandaging (MLLB) is the standard and recognised therapy for CDT. The advantage: the pressure is variable and bandages can be applied and reapplied so that as oedema is reduced the limb is reshaped.

CONCLUSION:

If you are seeing your patient regularly to apply the bandages or your patient is able to self-bandage, then MLLB may be the best solution to manage the oedema. However, if MLLB is not suitable, then JOBST® FarrowWrap®, a compression wrap system, could be the right solution.




JOBST® FarrowWrap®

Short-stretch wrap system is our new alternative product range especially for patients with fluctuating oedema, rebound oedema, donning and doffing problems or for patients who have limited access to professional bandaging. The overlapping bands provide support and rigidity to control oedema. The easy-to-use wrap system offers independence as it can be removed and reapplied by the patient.

CONCLUSION:

If your patient has a fluctuating oedema, difficulty in putting on a compression garment or is unable to access bandaging, then JOBST® FarrowWrap® might be the best solution.

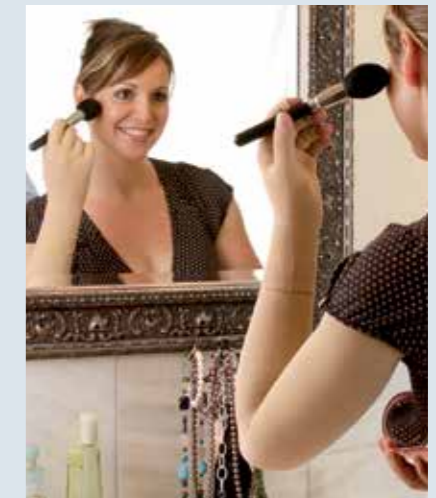













Flat knit garments


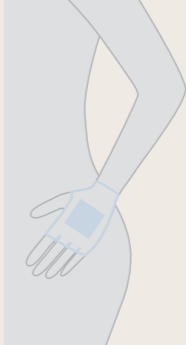
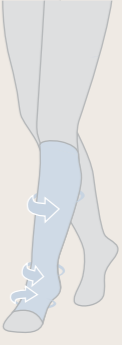
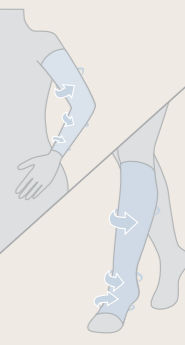
During, phase two of treatment known as the 'maintenance phase', compression garments are often considered the first choice. These are more aesthetically pleasing, allow normal clothing to be worn and have less layers than bandages or wrap systems. JOBST® Elvarex® for lymphedema management is a bespoke product range for the upper and lower limbs for your lower and upper extremities.

CONCLUSION:

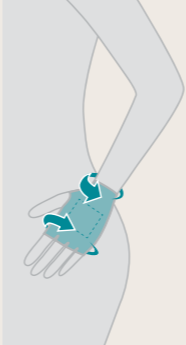If the oedema is stabilised and your patient needs a compression garment, then the best solution may be a custom-made, flat-knit JOBST® Elvarex® product. However, if a compression garment is not suitable, you may recommend JOBST® FarrowWrap®, a compression wrap system.

PRODUCT ASSORTMENT

	Footpiece	Legpiece	Foot Compression Liner	Thighpiece	Armpiece	Ambidextrous Hand Gauntlet Foam Squares
Mild to Moderate 20-30 mmHg						
JOBST® FarrowWrap® LITE	RTW	RTW		RTW	RTW	RTW
JOBST® FarrowHybrid® ADI			RTW			
Moderate to Strong 30-40 mmHg						
JOBST® FarrowWrap® STRONG	RTW	RTW		RTW		
JOBST® FarrowWrap® CLASSIC	RTW	RTW		RTW		
JOBST® FarrowHybrid® ADII			RTW			

	Hand Gauntlet Foam Squares	Legpiece	Arm- or Legpiece
Accessories			
JOBST® FarrowWrap® LITE	RTW		
JOBST® Farrow GarmentGrip™ Roll		RTW	RTW

JOBST® Farrow GarmentGrip can be cut to size to fit the shape and contour of the body and VELCRO® brand hook-and-loop fasteners can be added to secure in place. For triangular shaped legs, it is recommended to be worn over the legpiece to secure the wrap system in place.

JOBST® Farrow GarmentGrip is washable and therefore reusable. It is available for the upper and lower limb in differing roll lengths and extra wide widths and can also be used to manage mid-line oedema.

Ready to wear (RTW) pieces come in one convenient size to fit a wide range of patients.

RTW: Ready-To-Wear

UPPER EXTREMITY

JOBST® FarrowWrap® ARMPIECE – SUITABLE FOR 24-HOUR WEAR



LITE RTW Armpiece

LITE RTW Armpiece is an anatomically contoured effective solution to control upper extremity oedema. It is easy to apply and remove with one hand and designed for mild to moderate oedema control. Light and comfortable with just one layer but with a robust short-stretch performance.

- 20-30 mmHg compression

JOBST® FarrowWrap® HAND – SUITABLE FOR 24-HOUR WEAR



JOBST® FarrowWrap® LITE Hand Gauntlet 20-30mmHg

JOBST® FarrowWrap® LITE Hand Gauntlet is ambidextrous and comes with 4mm and 8mm foam squares for added compression to the dorsal hand. Suitable for mild to severe oedemas.

Benefits that give your patient greater freedom to enjoy life:

1. Foam inserts help reduce oedema that can pool on the top of the hand
2. Ambidextrous, i.e. can be used on either hand
3. Durable and comfortable

LOWER EXTREMITY

JOBST® FarrowWrap® THIGHPIECE AND LEGPIECE – SUITABLE FOR 24-HOUR WEAR

LITE Thighpiece



CLASSIC Thighpiece



STRONG Thighpiece



JOBST® FarrowWrap® LITE is designed for mild to moderate oedema control. LITE garments can be machine washed and dried*.

- 20-30 mmHg compression

JOBST® FarrowWrap® CLASSIC is designed for moderate to severe oedema control and resists popping into skin folds. CLASSIC must be hand washed*.

- 30-40 mmHg compression

JOBST® FarrowWrap® STRONG is designed for moderate to severe oedema control. STRONG garments can be machine washed and dried*.

- 30-40 mmHg compression



LITE Legpiece



CLASSIC Legpiece



STRONG Legpiece

* Follow care instructions in the product packaging.

LOWER EXTREMITY

JOBST® FarrowHybrid® LINERS



FarrowHybrid® is a patented foot compression garment that provides compression in the foot up to the ankle.

- From the ankle and above, it serves as a liner
- Available in **standard** and **wide** sizes
- Great alternative to a footpiece
- Low profile, easily fits into a shoe
- Available in two compression levels

	Small	Medium	Large	XLarge
Male Shoe Size	3-6	7-9	10-12	13-15
Female Shoe Size	4-7	8-10	11-13	14-16

Note the maximum calf size for Hybrid Liners is 60cm

Hybrid Liners

Measure the mid foot A circumference. Two mid foot sizes available: 20-27cm (regular foot) and 28-35cm (wide foot). Request the patient's shoe size (US Sizing).

JOBST® FarrowWrap® FOOT – SUITABLE FOR 24-HOUR WEAR



LITE Footpiece

LITE footpiece provides compression for mild to moderate oedema.

- 20-30 mmHg compression



CLASSIC Footpiece

CLASSIC footpiece is clinically proven in managing oedema. Durable construction.

- 30-40 mmHg compression



STRONG Footpiece

STRONG Footpiece provides compression for moderate to severe oedema. The long ankle bands can be wrapped around the ankle or over the forefoot to provide more compression where needed.

- 30-40 mmHg compression

Customising garments

On triangular shaped legs we recommend **JOBST® Farrow GarmentGrip™** be worn over our legpiece to help hold the garment in place.

HOW TO MEASURE

JOBST FarrowWrap® RTW Legpiece size chart

	XSmall	Small	Medium	Large	XLarge
Ready-to-wear pieces					
Legpiece					
C Calf	36-43 cm	42-50 cm	48-58 cm	53-63 cm	58-68 cm
B Ankle	21-25 cm	25-30 cm	30-36 cm	36-42 cm	42-50 cm
A-D Regular	35-39 cm	37-41 cm	39-43 cm	41-45 cm	42-46 cm
A-D Tall	40-44 cm	42-46 cm	44-48 cm	46-50 cm	47-51 cm

Length A-D = Floor to 2 finger widths below crease of knee. Measure posteriorly FOLLOWING contour of leg.

JOBST FarrowWrap® RTW Footpiece size chart

	XSmall	Small	Medium	Large	XLarge
Ready-to-wear pieces					
Footpiece					
A¹ Mid-foot	22-24 cm	25-27 cm	28-30 cm	31-34 cm	35-40 cm
X Regular	15-17 cm	16-19 cm	18-20 cm	19-22 cm	21-24 cm
X Long	18-20 cm	20-22 cm	21-23 cm	23-25 cm	25-27 cm

The length is measured from the most posterior aspect of the heel to the base of the 5th toe.

JOBST FarrowWrap® RTW Thighpiece size chart

	XSmall	Small	Medium	Large	XLarge
Ready-to-wear pieces					
Thighpiece					
G Groin	60-70 cm	65-75 cm	70-80 cm	75-90 cm	85-100 cm
E Above Knee	47-52 cm	53-59 cm	60-66 cm	67-73 cm	74-80 cm
E-G Short	16-20 cm	16-20 cm	16-20 cm	16-20 cm	16-20 cm
E-G Regular	21-26 cm	21-26 cm	21-26 cm	21-26 cm	21-26 cm
E-G Tall	27-31 cm	27-31 cm	27-31 cm	27-31 cm	27-31 cm

Length E-G = Measure **MEDIALLY** from just above patella to the groin. Follow outer contour of any skin folds/lobules.

LOWER EXTREMITY

JOBST® FarrowWrap® Kneepiece



Adjustable kneepiece, easy to apply & comfortable to wear.

Can be worn in combination with JOBST® FarrowWrap® Legpieces and Thighpieces, or as a single unit (e.g. in combination with an Elvarex knee high stocking)

Available in two fabrics: LITE and STRONG

Jobst® FarrowWrap® Kneepiece sizing chart

	Xsmall	Small	Medium	Large	X Large
5cm above patella	47 - 52cm	53 - 59cm	60 - 66cm	67 - 73cm	74 - 80cm
Calf	36 - 43cm	42 - 50cm	48 - 58cm	53 - 63cm	58 - 68cm




ORDERING INFORMATION




	Ref - No.	Size	Colour
JOBST® FarrowWrap® Lite (20-30mmHg) Kneepiece			
	73367-00	XSMALL	TAN
	73367-01	SMALL	TAN
	73367-02	MEDIUM	TAN
	73367-03	LARGE	TAN
	73367-04	XLARGE	TAN
JOBST® FarrowWrap® Strong (30-40mmHg) Kneepiece			
	73368-00	XSMALL	TAN
	73368-01	SMALL	TAN
	73368-02	MEDIUM	TAN
	73368-03	LARGE	TAN
	73368-04	XLARGE	TAN

ORDERING INFORMATION

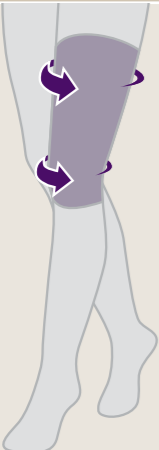


	Ref - No.	Size	Length	Colour
JOBST® FarrowWrap® Classic (30-40mmHg) Footpiece				
	76656-05	XSMALL	REGULAR	TAN
	76656-06	SMALL	REGULAR	TAN
	76656-07	MEDIUM	REGULAR	TAN
	76656-08	LARGE	REGULAR	TAN
	76656-09	XLARGE	REGULAR	TAN
	76656-00	XSMALL	LONG	TAN
	76656-01	SMALL	LONG	TAN
	76656-02	MEDIUM	LONG	TAN
	76656-03	LARGE	LONG	TAN
	76656-04	XLARGE	LONG	TAN
JOBST® FarrowWrap® Classic (30-40mmHg) Legpiece				
	76657-00	XSMALL	REGULAR	TAN
	76657-01	SMALL	REGULAR	TAN
	76657-02	MEDIUM	REGULAR	TAN
	76657-03	LARGE	REGULAR	TAN
	76657-04	XLARGE	REGULAR	TAN
	76657-05	XSMALL	TALL	TAN
	76657-06	SMALL	TALL	TAN
	76657-07	MEDIUM	TALL	TAN
	76657-08	LARGE	TALL	TAN
	76657-09	XLARGE	TALL	TAN
JOBST® FarrowWrap® Classic (30-40mmHg) Thighpiece				
	76658-06	XSMALL	SHORT	TAN
	76658-07	SMALL	SHORT	TAN
	76658-08	MEDIUM	SHORT	TAN
	76658-09	LARGE	SHORT	TAN
	76658-10	XLARGE	SHORT	TAN
	76658-00	XSMALL	REGULAR	TAN
	76658-01	SMALL	REGULAR	TAN
	76658-02	MEDIUM	REGULAR	TAN
	76658-03	LARGE	REGULAR	TAN
	76658-04	XLARGE	REGULAR	TAN
	76658-12	XSMALL	TALL	TAN
	76658-13	SMALL	TALL	TAN
	76658-14	MEDIUM	TALL	TAN
	76658-15	LARGE	TALL	TAN
	76658-16	XLARGE	TALL	TAN


ORDERING INFORMATION

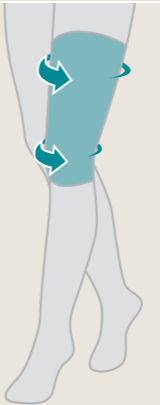	Ref - No.	Size	Length	Colour
JOBST® FarrowWrap® Strong (30-40mmHg) Footpiece				
	76646-04	XSMALL	REGULAR	TAN
	76646-05	SMALL	REGULAR	TAN
	76646-06	MEDIUM	REGULAR	TAN
	76646-07	LARGE	REGULAR	TAN
	76646-08	XLARGE	REGULAR	TAN
	76646-09	XSMALL	LONG	TAN
	76646-00	SMALL	LONG	TAN
	76646-01	MEDIUM	LONG	TAN
	76646-02	LARGE	LONG	TAN
	76646-03	XLARGE	LONG	TAN
JOBST® FarrowWrap® Strong (30-40mmHg) Legpiece				
	76648-00	XSMALL	REGULAR	TAN
	76648-01	SMALL	REGULAR	TAN
	76648-02	MEDIUM	REGULAR	TAN
	76648-03	LARGE	REGULAR	TAN
	76648-04	XLARGE	REGULAR	TAN
	76648-05	XSMALL	TALL	TAN
	76648-06	SMALL	TALL	TAN
	76648-07	MEDIUM	TALL	TAN
	76648-08	LARGE	TALL	TAN
	76648-09	XLARGE	TALL	TAN
JOBST® FarrowWrap® Strong (30-40mmHg) Thighpiece				
	76649-05	XSMALL	SHORT	TAN
	76649-06	SMALL	SHORT	TAN
	76649-07	MEDIUM	SHORT	TAN
	76649-08	LARGE	SHORT	TAN
	76649-09	XLARGE	SHORT	TAN
	76649-00	XSMALL	REGULAR	TAN
	76649-01	SMALL	REGULAR	TAN
	76649-02	MEDIUM	REGULAR	TAN
	76649-03	LARGE	REGULAR	TAN
	76649-04	XLARGE	REGULAR	TAN
	76649-10	XSMALL	TALL	TAN
	76649-11	SMALL	TALL	TAN
	76649-12	MEDIUM	TALL	TAN
	76649-13	LARGE	TALL	TAN
	76649-14	XLARGE	TALL	TAN

	Ref - No.	Size	Length	Colour
JOBST® FarrowWrap® Lite (20-30mmHg) Footpiece				
	76652-05	XSMALL	REGULAR	TAN
	76652-06	SMALL	REGULAR	TAN
	76652-07	MEDIUM	REGULAR	TAN
	76652-08	LARGE	REGULAR	TAN
	76652-09	XLARGE	REGULAR	TAN
	76652-00	XSMALL	LONG	TAN
	76652-01	SMALL	LONG	TAN
	76652-02	MEDIUM	LONG	TAN
	76652-03	LARGE	LONG	TAN
	76652-04	XLARGE	LONG	TAN
JOBST® FarrowWrap® Lite (20-30mmHg) Legpiece				
	76654-00	XSMALL	REGULAR	TAN
	76654-01	SMALL	REGULAR	TAN
	76654-02	MEDIUM	REGULAR	TAN
	76654-03	LARGE	REGULAR	TAN
	76654-04	XLARGE	REGULAR	TAN
	76654-05	XSMALL	TALL	TAN
	76654-06	SMALL	TALL	TAN
	76654-07	MEDIUM	TALL	TAN
	76654-08	LARGE	TALL	TAN
	76654-09	XLARGE	TALL	TAN
JOBST® FarrowWrap® Lite (20-30mmHg) Thighpiece				
	76655-05	XSMALL	SHORT	TAN
	76655-06	SMALL	SHORT	TAN
	76655-07	MEDIUM	SHORT	TAN
	76655-08	LARGE	SHORT	TAN
	76655-09	XLARGE	SHORT	TAN
	76655-00	XSMALL	REGULAR	TAN
	76655-01	SMALL	REGULAR	TAN
	76655-02	MEDIUM	REGULAR	TAN
	76655-03	LARGE	REGULAR	TAN
	76655-04	XLARGE	REGULAR	TAN
	76655-10	XSMALL	TALL	TAN
	76655-11	SMALL	TALL	TAN
	76655-12	MEDIUM	TALL	TAN
	76655-13	LARGE	TALL	TAN
	76655-14	XLARGE	TALL	TAN

HOW TO MEASURE

	Wrist	Elbow	Axilla
			
Small	14cm - 18cm	20cm - 27cm	22cm - 31cm
Medium	16cm - 21cm	25cm - 34cm	29cm - 39cm
Large	19cm - 25cm	30cm - 40cm	32cm - 45cm

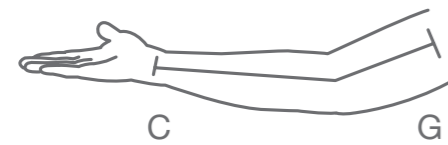


	Length
Short	40cm - 43cm
Regular	44cm - 47cm
Long	48cm - 51cm

NOTE: Measure length from the outside of the wrist to the axilla (G) with a slight bend in arm



ORDERING INFORMATION

	Ref - No.	Size	Option	Colour	
JOBST® FarrowWrap® Lite (20-30mmHg) Armpiece Long					
	76650-35	SMALL	LEFT	TAN	
	76650-00	SMALL	RIGHT	TAN	
	76650-01	MEDIUM	LEFT	TAN	
	76650-02	MEDIUM	RIGHT	TAN	
	76650-03	LARGE	LEFT	TAN	
	76650-04	LARGE	RIGHT	TAN	
	76650-17	SMALL	LEFT	BLACK	
	76650-18	SMALL	RIGHT	BLACK	
	76650-19	MEDIUM	LEFT	BLACK	
	76650-20	MEDIUM	RIGHT	BLACK	
	76650-21	LARGE	LEFT	BLACK	
	76650-22	LARGE	RIGHT	BLACK	
JOBST® FarrowWrap® Lite (20-30mmHg) Armpiece Regular					
	76650-05	SMALL	LEFT	TAN	
	76650-06	SMALL	RIGHT	TAN	
	76650-07	MEDIUM	LEFT	TAN	
	76650-08	MEDIUM	RIGHT	TAN	
	76650-09	LARGE	LEFT	TAN	
	76650-10	LARGE	RIGHT	TAN	
	76650-23	SMALL	LEFT	BLACK	
	76650-24	SMALL	RIGHT	BLACK	
	76650-25	MEDIUM	LEFT	BLACK	
	76650-26	MEDIUM	RIGHT	BLACK	
	76650-27	LARGE	LEFT	BLACK	
	76650-28	LARGE	RIGHT	BLACK	
	JOBST® FarrowWrap® Lite (20-30mmHg) Armpiece Short				
		76650-11	SMALL	LEFT	TAN
		76650-12	SMALL	RIGHT	TAN
		76650-13	MEDIUM	LEFT	TAN
76650-14		MEDIUM	RIGHT	TAN	
76650-15		LARGE	LEFT	TAN	
76650-16		LARGE	RIGHT	TAN	
76650-29		SMALL	LEFT	BLACK	
76650-30		SMALL	RIGHT	BLACK	
76650-31		MEDIUM	LEFT	BLACK	
76650-32		MEDIUM	RIGHT	BLACK	
76650-33		LARGE	LEFT	BLACK	
76650-34		LARGE	RIGHT	BLACK	

SELECTING THE RIGHT PRODUCT FOR YOU AND YOUR LYMPHOEDEMA PATIENTS

BSN medical focuses on integrated therapy solutions, offering an extended variety of innovative products to meet the needs of patients in all therapy stages. JOBST® FarrowWrap® is versatile and easy to use, and offers an alternative in the treatment of lymphoedema patients.



DECONGESTION

Therapy stages		LATENT Stage 0	MILD Stage 1	MODERATE Stage 2	SEVERE Stage 3
COMPRESSION PRODUCTS	WRAP SYSTEMS		JOBST® FarrowWrap® LITE		
				JOBST® FarrowWrap® STRONG	
				JOBST® FarrowWrap® CLASSIC	
	BANDAGES			JOBST® Compri2	
				Comprilan®	



MAINTENANCE

Therapy stages		LATENT Stage 0	MILD Stage 1	MODERATE Stage 2	SEVERE Stage 3
COMPRESSION PRODUCTS	CUSTOM GARMENTS		JOBST® Elvarex®		
			JOBST® Elvarex® Plus		
			JOBST® Elvarex® Soft		
			JOBST® Elvarex® Soft Seamless		
	READY-TO-WEAR	JOBST® Bella™ Lite			
			JOBST® Bella™ Strong		
	WRAPS		JOBST® FarrowWrap® LITE		
				JOBST® FarrowWrap® STRONG	
				JOBST® FarrowWrap® CLASSIC	

BSN medical Australia & New Zealand

L6, 211 Wellington Road,
Mulgrave, Victoria, Australia 3170

www.bsnmedical.com.au
www.bsnmedical.co.nz

BSN0029v2 03/2018

JOBST® Customer Service Australia
T 1300 998 810 F 1300 998 820

JOBST® Customer Service New Zealand
T 0508 998 810 F 0508 998 820



an Essity company