

Swivel Shower Chair

Swivel Bather is a registered trade mark of Myco Limited

User Instructions



Swivel Shower Chair Overview

The Ultra Swivel Shower Chair is supplied with the Locking Lever on the right hand side of the frame. The lever can be changed to the left hand side of the frame by removing the pivot bolt, changing the side of the lever, then reinserting the bolt in the appropriate place.


The Ultra Swivel Shower Chair should be cleaned regularly with a solution of mild detergent and household disinfectant in warm water. Do not use abrasive cleaning products.

Seat height adjustment from 18½" to 21½" (470mm to 550mm)

Maximum user weight 25 stone (160Kg)

Assembling the Swivel Shower Chair

- Position the Locking Lever so that it is on the right hand side of the seating unit (unless changed to left hand side - see paragraph 1).
- The socket under the seat locates over the swivel pin of the base. Press down on the seat so that the seat unit is fully over the swivel pin and touching the base. Ensure that the seat rotates freely.
- The seat will lock at each 90° interval as the locking lever drops into position. Lift the Locking Lever to swivel the seat.

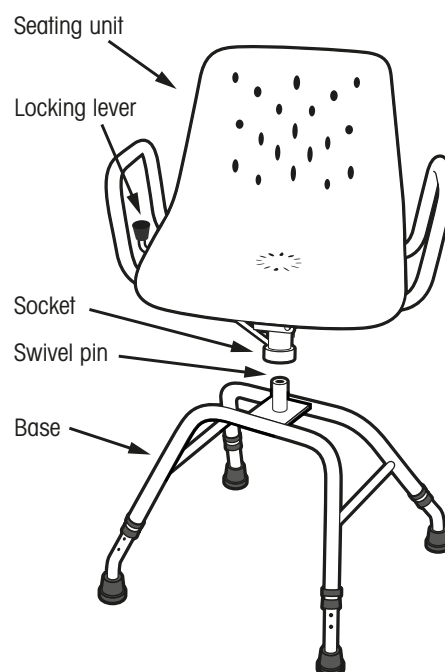

Adjusting the height of the Swivel Shower Chair

Adjust one leg at a time:

- Remove the C-Clip on the leg by pushing the open side and pulling out the pin.
- Slide the leg to the height required – and adjust until the holes in the adjustable leg align with the hole in the outer leg.
- Replace the C-Clip. Ensure that the pin goes through both holes in the outer leg and the plastic ring is locked in position.
- Check that all four legs are at the same height and that the chair is stable.

