

Patient Care Products a world of medical needs

Pedal Exerciser-24341

This product will last for many years if used in a safe and correct manner. The following information is supplied as a guide only. We recommend the correct usage, fitting and suitability of the product be advised by a <u>OUALIFIED THERAPIST</u>.

INSTRUCTIONS

Distributor

- Examine the product for possible shipping damage
- Ensure customer has been assessed & advised by a health professional
- Ensure customer has been briefed thoroughly about product capabilities & correct usage.

User

- · Before you undertake any program of exercise please consult with your health professional.
- · Please keep all children away from exercise products.
- Ensure there is at least 1 meter of free space in all directions around your product whilst you are exercising.
- Ensure your product is positioned on a solid, flat surface. If necessary, user a rubber mat underneath to reduce the possibility of slipping.
- Always wear appropriate clothing & footwear such as training shoes when exercising. Do not wear loose clothing.
- Do not use your exercise product whilst standing.

MAINTENANCE

- Regularly check all moving parts for obvious signs of wear & tear.
- Regularly check that there are no defects in the frame. If a defect appears, do not attempt to use the
 product & contact your supplier immediately for advise.
- Clean with a dry cloth. Do not use solvent cleaners.

WARRANTY CONDITIONS

PCP offers a conditional 12 month warranty against faults in materials and manufacturing. PCP will replace or repair the product at their discretion if deemed under warranty.

- 1. 12 months conditional warranty (Limited usage Domestic)
- 2. Provide copy of receipt of purchase.
- 3. Product to be returned to distributor.
- 4. Does not cover shipping costs incurred in replacement or repair.
- 5. Does not cover dents, scratches or blemishes which occur after purchase date.
- 6. Does not cover consumable items subject to "wear and tear".
- 7. Does not cover user negligence, accident or improper installation.
- 8. Does not cover unauthorized repairs or alterations.
- 9. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.
- 10. Warranty does not cover products used for hire.

www.patientcareproducts.com.au